


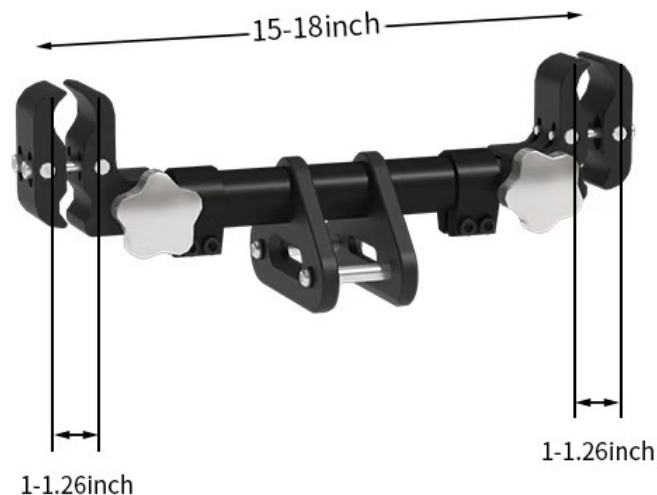

The diameter of the round tube shall be **1-1.26inch**, and the distance between two round tubes of the wheelchair shall be **15-18inch**.



The round tube is not fixed



There is no position available for clamping



Note: The adapted wheelchair must ensure that there is a round tube with a space of more than **1.57inch** at this position for the accessories to clamp. The clamped round tube must be fixed and not slide to ensure safety.



Optional Brake Lever (Non-Mandatory Installation): Customers can choose to install it as needed. If the wheelchair's built-in brake lever does not conflict with the horizontal axis installation, installation can be omitted.

Three Installation Positions Available



Forward Assembly



Reverse Assembly

(Assembly method shall be determined according to actual conditions.)